

# Memo

To: AML Reference Laboratory Clients  
From: Allina Medical Laboratories, Reference Laboratory Services  
Date: June 13, 2007  
Re: AQU No Longer Available

---

Effective immediately, the 322/AQU, Amino Acid Fractionated – Urine Quant, is no longer available. The alternate test is the 291/AAU, Amino Acid Screen – Qualitative Urine. Test information for the alternate test is as follows:

**Test Name:** AMINO ACID SCREEN - QUALITATIVE URINE  
**Test Number:** 291/AAU  
**Collect:** 10 ml Urine - Random -No Preservative  
**Peds Collect:** 2 ml Frozen Urine Aliquot  
**Container:** Screw Cap Urine Container  
**Processing:** Submit 10 ml Frozen Aliquot of Urine  
**Transport/Stability:** Frozen  
**Alternate Names:** INBORN ERRORS OF METABOLISM SCREEN  
**Performing Lab:** Quest Diagnostics (#8730N)  
**Days Set Up:** M - F (AM)  
**Expected TAT:** 2 - 3 Days  
**Ref. Ranges:** See Report  
**Collection/  
Processing Details:** RANDOM OR 24 HOUR SPECIMEN. If 24 hours record Total Urine Volume on request and / or aliquot. Provide Age, Sex, tentative Dx and drug history. Remember to leave expansion room in the Container for freezing to prevent leaking of specimen.  
**Method:** HPLC  
**CPT Codes:** 82128-90

If you have any questions regarding this message, please contact your AML Account Representative.