

# Special Collection and Processing Instructions

**Label Instructions**

**On-line Collection Manual Instructions to be Used by Ref. Lab clients**

12760000

ssn 2000031934 XDEM  
 epic n/a REFLAB  
 TEST, FELIXTHECAT  
 DoB 01/01/01 F ANW-CA  
 MRN:n/a 05/08/06 13:51  
 By Time  
 HOM



ANW **LT GREEN**  
 Draw on ice.

HOMOCYSTEINE---PST Tube - Spin, Separate, Refrig within 30 minutes. SST tube also acceptable - Clot on ice or in refrigerator for 60 min; Spin, separate and refrigerate.

12759999

ssn 2000031934 XDEM  
 epic n/a REFLAB  
 TEST, FELIXTHECAT  
 DoB 01/01/01 F ANW-CA  
 MRN:n/a 05/08/06 13:51  
 By Time



ANW **LT GREEN**  
 On ice, do not open.

AMMONIA---PST Tube - Place on ice after collection. Spin, Separate (into plastic transfer vial) and Freeze Immediately.

12353115

ssn 2000031934 XDAN--  
 epic n/a REFLAB  
 TEST, FELIXTHECAT  
 DoB 01/01/01 F ANW-CG  
 MRN:n/a 03/24/06 11:00  
 By Time




ANW **LT GREEN**  
 On Ice/DON'T Open or Spin

IONIZED CALCIUM---SST (required)-Spun Unopened - Serum must be spun within 1 hour of collection. DO NOT OPEN. Serum stable refrigerated 7 days. Heparinized whole blood specimens will be **REJECTED** due to short stability.( 30min at Room Temp, 4 hours Refrigerated.)

12760001

ssn 2000031934 XDEM  
 epic n/a REFLAB  
 TEST, FELIXTHECAT  
 DoB 01/01/01 F MSO-SO  
 MRN:n/a 05/08/06 13:51  
 By Time

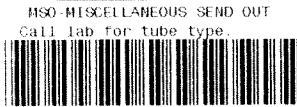

MSO **CALL LAB**  
 Call lab for tube type.

MISCELLANEOUS SEND OUT---See AML On-line Collection Manual for instructions.